EN

Name
BAPSCARCARE gel SPF 40

Product description
BAPSCARCARE gel SPF is a silicone gel, enriched with vitamin E for the treatment and prevention of abnormal scars (hypertrophic and keloid) while protecting against harmful UVA and UVB rays (SPF 40).
BAPSCARCARE gel SPF can be applied to any part of the skin and is particularly suitable for treating scars on the face, neck, at joints and in skin folds.

Composition
Polyolsioxanen, Caprylyl/Capric Triglyceride, Titanium dioxide, Cyclopentasiloxane, Zinc Oxide, Polyhydroxyestearic acid, Alumina, Stearic acid, Triethoxycaprylylsilane, Vitamin E.

Properties
-BAPSCARCARE gel SPF makes scars flatter, more flexible and thinner and reduces complaints (such as persistent redness, scar pain, itching, discoloration of joint movement, discomfort and cosmetic concerns).
- The gel provides a semi-occlusive (covering) layer on the surface of the skin, resulting in increased hydration of the skin (epidermis). This increased hydration accelerates the recovery of damaged scar tissue and reduces complaints.

Indications/areas of application
BAPSCARCARE gel SPF provides combined therapy in the form of both scar treatment and sun protection and can be applied to both old and new scars for:
- the treatment and prevention of abnormal scars (hypertrophic and keloid),
- the treatment of burn scars,
- the treatment of surgical scars,
- the treatment of persistent functional issues caused by the scar (persistent redness, scar pulling, itching, restriction of joint movement, discomfort and cosmetic concerns).

Sun protection factor SPF 40
Sunlight is healthy, but can also be potentially dangerous. There are 2 types of sunlight: UVB and UVA rays. [Excessive] exposure to UVB rays can lead to sunburn. Excessive exposure to UVA rays accelerates skin ageing and can cause hypersensitivity to the sun and hyperpigmentation (pregnancy mask, freckles). Both types of radiation can affect cellular DNA and contribute to the development of skin cancer in the long term. Scars should be protected from the sun's harmful rays at all times. With BAPSCARCARE gel SPF you can prevent or treat a scar while protecting against harmful UVA and UVB rays.

Instructions for use
For optimum results, preferably start treatment with BAPSCARCARE gel SPF immediately after wound closure (i.e. after removal of sutures and disappearance of any scabs).
- Twist the cap of the tube and remove the film from the tube.
- Clean the skin before use and allow it to dry.
- Apply thinly twice a day, a white layer may remain visible. This can be rubbed in, as required.
- Replace the cap on the tube after use.
- Reapply BAPSCARCARE gel SPF after intensive activity (sports) and contact with water (showering, swimming).
- If desired, BAPSCARCARE gel SPF can be combined with make-up as soon as the gel has dried up.

When exposed to the sun
- Apply well in advance.
- Reapply regularly (every 2 hours). Especially after swimming, sports and drying the skin.
- Avoid the intensive midday sun (between 11:00 and 15:00).
- Wear clothing that covers the skin, for extra protection: hat, T-shirt and sunglasses.
- Always keep babies and young children out of full sunlight.

For how long should I use BAPSCARCARE gel SPF?
- It takes at least 3-6 months for a scar to normalise.
- Use BAPSCARCARE gel SPF as long as the scar is elevated, red and/or visible.
- Consult your therapist/doctor on the duration of the therapy.

Duration of use
- When used correctly, a tube is sufficient for treatment lasting a few weeks.
- The duration of use depends on the size of the scar to be treated and the amount of sun exposure.
- In the case of long-term therapy, several packs may therefore be required.

Special attention and precautions
- Keep out of the reach of children.
- Avoid contact with eyes and mucous membranes.
- The silicone gel is only suitable for external use.
- Do not use the silicone gel on healed, closed skin (after the scab has disappeared) and never on open wounds.
- Do not use in case of hypersensitivity to any of the ingredients.
- Do not use if the packaging is damaged on receipt of the product.

Side effects
- In rare cases, the skin may react sensitively (skin irritation) to silicone treatment. In case of severe skin irritation, stop use.
- If necessary, BAPSCARCARE gel SPF can be removed using a mild, skin-friendly soap.
- Stop the treatment again as soon as the skin has recovered from irritation.
- If skin problems persist, consult your therapist/doctor.
- If there are any serious incidents related to the product these should be reported to the manufacturer and the competent authorities of the Member State in which the user resides.

Storage
- The expiry date is stated on the packaging.
- Can be kept for a maximum of 6 months after opening.
- Store at room temperature or at least not at temperatures higher than 25°C.

Date of last revision of the instructions for use
January 2021

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